

Technical specifications

Intended use	Endodontics						
Indications for use	Retro preparation of cavities and isthmuses						
Description	Diamond-coated right-angled retro tip for preparing pre-molar and molar canals						
Tip type	Smooth					~	
Allergenic factor	Tip made of surgical stainless steel as per ASTM F899 - 12b						
Aller geriic ractor	Surgical stainless steel contains: • nickel • chromium						
Commercial reference	P15RD: F00108						
Medical class	Class of medical device: IIa according to 93/42/EEC directive						
Adjustments	Newtron range	P5 Range	Suprasson P5 Booster		y Max - P-Max - nter 2 - Piezotome 2	Irrigation	
	7	7	5	3		^	

Service life

As it is not possible to establish a maximum number of uses (that may be determined by many parameters such as duration of use, hardness of enamel, the force applied, wear), we recommend that medical devices being routinely used are replaced at least once a year. Replace the medical device if the tip does not vibrate at the expected frequency, the treatment is not progressing as normal and is taking longer or is at a standstill.


Associated documentation

Document title	References
Consulting electronic user instructions	J00007
Cleaning, disinfection and sterilization instructions for tips	J02001
Apical Surgery clinical tips	J08061
Ultrasonic generator power settings table	J58000
General instructions relating to the complete range of standard ultrasonic dental tips	J02101


Sterilization




Sterilize prior to each use

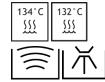


The tips must be cleaned, disinfected and sterilized prior to each use. Prior to each use, please refer to the cleaning, disinfection and sterilization instructions for tips J02001.

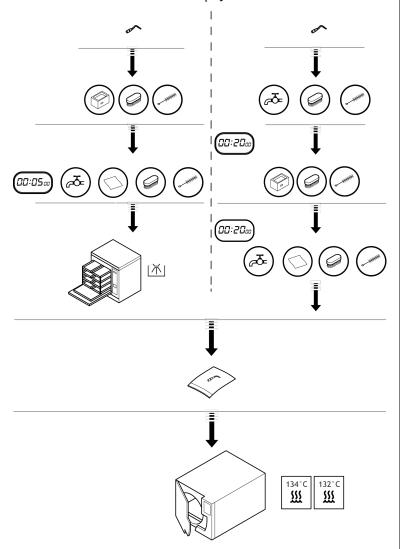

Installation

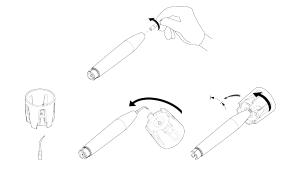

Disassembly

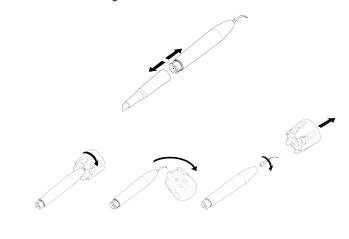

Manufactured For:



EdgeEndo 5600 Wyoming Blvd NE Suite 100 Albuquerque, NM 87109

www.edgeendo.com



Date of first CE marking

1998

Tip | P15RD | J00421 | (98) | V1 | 03/2017 | NB18EN020A